

Syntermed Installation Guide

If the computer where the installation is taking place will have an active Internet connection, an activation code may be requested which will allow MCP to be licensed without the use of an offline license.

If an active Internet connection will not be available, the MCP component of Syntermed's software suite must be installed before a license can be generated. Syntermed needs the machine specific Secure Code that is displayed the first time our software is run to create the license.

1. Download Syntermed software packages from the links provided.
2. Install MCP and Emory Cardiac Toolbox from the .zip files.
3. When completed, double click the newly created Syntermed icon on your Desktop. This will launch MCP and you will be prompted for an Activation Code. Select "off-line activation" from the Activation Code window. A "Secure Code" will be presented, forward that with the other information requested in the form found under Licensing Procedure.
4. Once approved, Syntermed Support will provide a machine specific license file. Save this file to a known location on installation system, "browse..." to it from the Secure Code window.

Nuclear is evolving. We're evolving with you.